# Instructions for use

CTi® OTS

*Life Without Limitations®*

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	9
ⓕⓗ Guide de fabrication.....	13
ⓔⓢ Instrucciones para el uso.....	17
ⓔⓣ Istruzioni per l'uso.....	21
Ⓝⓞ Bruksanvisning.....	25
ⓓⓐ Brugsanvisning.....	28
Ⓢⓕ Bruksanvisning.....	32
ⓕⓙ Käyttöohjeet.....	36
Ⓝⓕ Gebruiksaanwijzing.....	40
ⓔⓣ Instruções de Utilização.....	44
ⓙⓐ 取扱説明書.....	48

EN - Caution: Ossur products and components are designed and tested to ISO 10328. Compatibility and compliance with this standard is achieved only when Ossur products and components are used with other recommended Ossur or authorized components. If unusual movement or product wear is detected in a structural part of a prosthesis at any time, the patient should be instructed to immediately discontinue use of the device and consult his/her clinical specialist.

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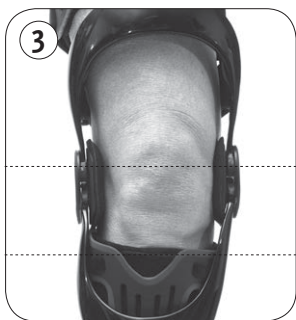
## SIZING CHART

Size	Caliper Measurement*	
Small	9 - 10 cm	3 1/2" - 4"
Medium	10 - 11,5 cm	4" - 4 1/2"
Large	11,5 - 12 cm	4 1/2" - 4 7/8"
X Large	12 - 13 cm	4 7/8" - 5 1/4"
XX Large	13 - 14,5 cm	5 1/4" - 5 3/4"

\* Caliper measurement (M-L at joint space) should be taken in standard weight bearing position.

## INDICATION FOR USE:

Anterior Cruciate Ligament instability, posterior cruciate ligament instability, medial collateral ligament injury, lateral collateral ligament injury. Low, medium and high contact/impact activities.

## WARNING

This Knee Brace is a supportive brace only and is not intended or guaranteed to prevent knee injury. Purchaser agrees to the following:

1. Ossur accepts no responsibility for any injury which might occur while using Ossur knee bracing products.
2. With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If you experience any unusual reactions while using this product, consult your health care provider immediately.
3. If you have any questions as to how this knee brace applies to your own specific knee condition, consult your health care provider. If you do not agree to the above conditions, contact your brace provider before using this product.

## BRACE APPLICATION

1. Unfasten all straps from the outer side of the brace.
2. With the knee bent in approximately 90 degrees, place the brace over your knee such that the patella is centered between the condyle pads (**figure 2**)  
**NOTE:** This places the hinge slightly above the joint line so the brace can properly seat itself when you stand. (**figure 3**)
3. Beginning with the strap just below the knee (lower cruciate strap), fasten the strap snugly above the calf muscle as this will aid in the brace suspension. (**figure 4**)
4. Next, snugly fasten the bottom strap by inserting it through the D-ring on the subshell.
5. Continue with the strap just above the knee (upper cruciate strap), followed by the top strap. Snugly fasten both straps.  
**NOTE:** Over-tightening the two top straps can cause the brace to migrate.

**WARNING:** If brace causes discomfort or irritation, discontinue use immediately and contact your health care provider.

## **BRACE MAINTENANCE**

The CTi OTS brace is designed for minimum maintenance and care. Follow these easy maintenance steps to keep your brace in good condition.

### *Cleaning*

- Liners, straps and pads may be removed for cleaning.
- Do NOT machine wash
- Do NOT place in dryer
- Hand-wash using mild non-detergent soap
- Rinse thoroughly
- Air-dry,

If migration occurs, clean silicone liners with an alcohol wipe or damp cloth. If brace is used in salt water or chlorinated water, rinse well and air dry.

### *Hinge maintenance*

- Do NOT disassemble the hinge yourself
- Check regularly for foreign materials in hinge (i.e. dirt or grass)
- Clean dirt from hinge using water or compressed air
- Lubricate with silicone spray or dry graphite if needed

## **ACCESSORIES**

The following accessories are available for your CTi OTS brace at an additional charge.

To order call your Ossur Customer Service Department.

- Patellar cup - Knee cap protection, especially for motocross, mountain biking or work that requires kneeling.
- Flexion stop kit - Stops can be set from 0° - 90°
- Anti migration wrap
- PCL Kit - to counteract against posterior subluxation of the tibia

## **SETTING UP THE PCL SUPPORT SYSTEM**

The PCL kit is designed to counteract against a posterior subluxation of the tibia. The thigh plate and strap apply a posterior force against the anterior thigh, while the lower straps and calf plate apply an anterior force to the tibia. To place the PCL system in the correct position on the leg:

1. Apply the brace as described earlier with the anterior thigh plate and straps loose.
2. Attach PCL cuff to the lower straps and center over the calf muscle. Tighten straps so that the cuff fits comfortably but snugly against the back of the leg.
3. Tighten anterior thigh strap with the leg in full extension so that the plate and pad are comfortably snug.

## **REFURBISHING**

From time to time, your CTi OTS should be sent to Ossur for refurbishing. The frequency of this will depend on how often the brace is worn and for what activities. Your brace supplier should help you to determine when your brace needs to be sent in for service, but once a year is a good rule of thumb.

As part of the refurbishment process, Ossur will check your brace for signs of wear, repaint it, and rebuild it with all new components, including rivets, buckles, straps and padding. There is a usual turnaround time of three to five working days. If you have any questions, contact your local Customer Service Department.

## **FIT ADJUSTMENTS**

### *Widening or narrowing fit at knee joint:*

The CTi OTS brace can be adjusted for individual knee width. To widen

or narrow the width at the knee, the 6 mm (1/4") padding located under each cover can be removed and replaced with thinner pads [3 mm ("1/8) or 1,5 mm ("1/16)] or you can install a shim behind the 6 mm (1/4") pad to make the fit more snug. Any combination of pads can be used to achieve a more comfortable fit.

To accommodate subtle variances in the plane of the patient's leg, the anterior tibial strap (ATS) (**figure 5**) should be loosened or tightened as needed so that the center of the hinges is slightly posterior of the midline. In conjunction, adjust the lower strap and the bottom strap. If the brace is too far anterior, loosen the tibial alignment strap and tighten the lower and bottom straps. If the frame is too far posterior, loosen the lower and bottom straps AND tighten the tibial control strap. Accurate planar setting will help to ensure optimal brace suspension.

#### *Adjusting fit at the tibial horseshoe:*

Tibial wedge pads can be installed to fill in any gaps around the tibia (**figure 6**) and achieve the desired fit. To add a wedge pad, pull back the silicone liner and place the proper wedge pad (medial or lateral) on top of the hook-and-loop fastener to which the silicone liner was originally fastened (**figure 7**). Press the silicone liner back into place on top of the wedge pad. Fine tune the padding until your CTi OTS brace fits snugly against the tibia (**figure 8**).

### **CHANGING EXTENSION STOPS**

*Extension stops are available for varying degrees of extension control.*

1. Remove the cover and pad from the condylar area.
  2. Using a short Philips head screwdriver, remove the screw in the center of the hinge plate (**figure 9**).
  3. Locate the extension stop positioned in the hinge between the upper and lower arms of the brace. With brace slightly bent, push the extension stop from the back of the brace through to the front. This should cause the stop to pop out of the hinge (**figure 10**).
  4. From the bag of extension stops included with the brace, choose the stop which corresponds to the desired degree of extension. Feed the hole end of the stop through the hinge from the front to the back of the brace, with the curved side facing down and the flat edge facing up. Make sure that the sharp extrusions on the flat end of the extension stop are pointed toward the distal portion (bottom half) of the brace (**figure 10**). The flat end of the stop should be visible from the front of the brace.
  5. Align the hole in the center of the hinge plate with the hole in the extension stop. Replace and tighten the screw through the hinge plate and extension stop. Replace the pad and cover. This procedure must be completed on both the medial and lateral hinges.
  6. Choose the appropriate extension stop (Blue = 0°, Black = 0-10°, Dark Gray = 10-20°, Medium Gray = 20-30°, Light Gray = 30-40°)\*
- These numbers are approximate. Actual extension stop will vary from person to person depending on anatomical differences. Appropriate extension stop size is normally determined by health care provider.

### **LIMITED WARRANTY**

This warranty covers all defects in materials and workmanship of your new CTi OTS Knee Brace. This Warranty covers the rigid carbon fiber brace frame and Accutrac™ hinges for 2 years. The warranty covers all other materials, including straps, sleeves, liners and pads for six months from the date of original purchase.

Ossur will repair or replace (at its option) any defective part covered under warranty at no charge. Repairs necessitated by accident, negligence, misuse or abuse by the purchaser are not covered. Also, consequential and incidental damages are not recoverable under this warranty. These limitations may not apply as some states do not allow the exclusion or

limitation of incidental or consequential damages. Only the original purchaser is entitled to claim service under this warranty.

To obtain service under this warranty call our Customer Service Department.

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CTi® OTS is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.