# Instructions for use

Unloader<sup>®</sup> One  
OTS & Custom

*Life Without Limitations<sup>®</sup>*

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	8
ⓕⓗ Guide de fabrication.....	11
ⓔⓢ Instrucciones para el uso.....	14
ⓔⓣ Istruzioni per l'uso.....	17
Ⓝⓞ Bruksanvisning.....	20
ⓓⓐ Brugsanvisning.....	23
ⓔⓑ Bruksanvisning.....	26
ⓕⓙ Käyttöohjeet.....	29
Ⓝⓕ Gebruiksaanwijzing.....	32
ⓔⓣ Instruções de Utilização.....	35
ⓙⓐ 取扱説明書.....	38
ⓙⓗ 中文 – 注意.....	41
ⓙⓞ 사용 설명서.....	43

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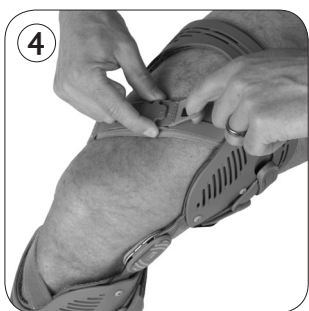
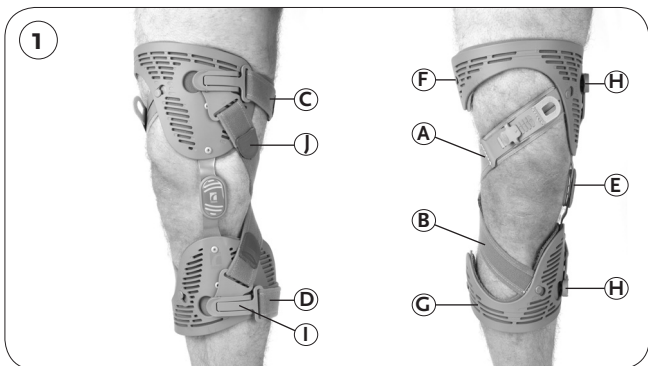
## UNLOADER® ONE INDICATION FOR USE:

- Mild to Severe Unicompartmental Osteoarthritis
- Unicompartmental knee conditions that require load reduction to affected compartment such as:
  - Articular cartilage defect repair
  - Meniscal cartilage repair
  - Avascular necrosis
  - Tibia plateau fracture
- Normal activities of daily living
- Circumference limitation
  - Thigh circumferences smaller than 13.5" (34.3 cm) or bigger than 29" (73.5cm).
  - Calf circumferences smaller than 10" (25 cm) or larger than 24" (61.5cm).

**CAUTION:** For patients with peripheral vascular disease, neuropathy, and sensitive skin, close physician supervision is recommended.

The Unloader® One pain relief device is field adaptable to accommodate patient comfort issues. Modifications to flexion or extension stops and shell contour should only be made by the health care professional. The healthcare professional is responsible for advising patient on use of the Dynamic Force Control System system.

**NOTE:** Have patient wear brace for a few minutes to make sure straps are at comfortable tension.

## DONNING BRACE

1. Align brace on patient's leg with center of hinge at mid patella and midline of a / p plane (**Figure 1**).
2. With both Quick Fit Buckles open - place button of lower Quick Fit Buckle into appropriate keyhole above buckle-stability pin (**Figure 2**) and close carefully (**Figure 3**). The button is now color matched with an identical blue color swatch in the keyhole for intuitive strap selection.
3. Bend knee to an 80° angle - foot flat on floor.
4. Place button of upper Quick Fit Buckle into appropriate keyhole (yellow) and close carefully.
5. Tighten Dynamic Force Control System to a prescribed position (**Figure 4**): Practitioner will need to determine optimum unloading based on patient's pain relief feedback but initial setting is recommended at 5.
6. Fully extend leg in a sitting position and make sure that brace is still aligned correctly on leg.
7. Ensure strap pad is not wrinkled and is positioned under strap crossing in popliteal. Quick Fit Buckles may have to be opened to relieve strap tension and allow positioning of strap pad on non-ratcheted DFS strap (**Figure 13**).
8. If patient requires more or less unloading, adjust the tension of upper DFS strap accordingly with Dynamic Force Control System and/or lower DFS strap by adjusting length at thigh Quick Fit Buckle.
9. If Dynamic Force Control System has been pulled to its maximum and patient requires more pain relief, DFS straps need to be shortened accordingly on lower Quick Fit Buckle.

10. Keyhole locks may be inserted into calf and thigh shells (**Figure 5**) if desired to ensure straps cannot accidentally be disengaged from shells.

**NOTE:** Have patient wear brace for a few minutes to make sure straps are at comfortable tension. Pressure reducing pads are included and may be applied for additional comfort.

### REMOVING BRACE

1. Press button on Dynamic Force Control System to release tension in ratcheted- DFS strap (**Figure 6**).
2. Open both upper and lower Quick Fit Buckle
3. Remove Quick Fit Buckles from keyholes
4. Remove brace from leg

### ONE-TIME STRAP LENGTH ADJUSTMENT (OPTIONAL):

1. Remove alligator clips (J) and trim straps. DFS Alligator clips should be positioned nearby to buckles (**Figure 7**) away from popliteal region.
2. Repeat for all straps.

### TAMPER PROOF STRAP LENGTH SET-UP:

The brace now ships with straps folded externally onto themselves for easy adjustment. If desired, the straps may be reversed and folded in an internal termination for a tamper proof patient configuration:

1. Remove alligator clips.
2. Pull strap out of D-ring/buckle and thread through D-ring/buckle in a reversed manner, from outside to inside (**Figure 8**) .
3. Use double sided hook provided to secure strap end (**Figure 8**).
4. Repeat for all straps.

### RANGE OF MOTION ADJUSTMENT (ROM):

The device comes with a 0° extension stop installed. Additional extension stops of 5°, 10°, 15° and 30° have been provided. A flexion stops of 90° is also included.

**NOTE:** Optimal corrective forces are applied at full extension. If patient requires an extension stop change, make sure to readjust tension in DFS straps if more unloading is required.

### TO CHANGE RANGE OF MOTION:

1. Peel condyle pad from hinge to expose screw.
2. Remove retaining screw.
3. Slide 0° stop out of hinge.
4. Insert desired extension stop.
5. Secure with retaining screw (**DO NOT OVER TIGHTEN**)
6. For adding flexion stop, follow steps 1, 4 & 5.

### FINAL INSPECTION: Right leg brace

Figure 9. Front

Figure 10. Medial side

Figure 11. Lateral side

Figure 12. Back

### BRACE MAINTENANCE

Unloader® One pain relief device is designed for minimum maintenance and care. Follow these easy maintenance steps to keep brace in good condition.

### CLEANING

- Liners may be wiped down in place with alcohol for a quick cleaning.
- For more thorough cleaning, liners, straps and strap pads may be removed.
- **DO NOT MACHINE WASH.**
- **DO NOT PLACE IN DRYER**
- Hand-wash using mild non-detergent soap.
- Rinse thoroughly.

- Air-dry.
- Strapping Sequence (**Figure 13**).
- Quick fit buttons, buckles, and shells have now been labeled with matching symbols to ease reassembly (**Figures 14 and 15**).

If migration occurs, clean liners with an alcohol wipe or damp cloth.

#### **HINGE**

- For safety reasons, hinge should only be disassembled by a health care professional.
- Check regularly for foreign materials in hinge i.e. (dirt or grass).
- Clean dirt from hinge using tap water or compressed air.

If brace is used in salt water or chlorinated water, rinse well and air dry brace. A regular service routine will ensure hinge, liners, straps and pads are in optimum working order.

#### **BRACE ACCESSORIES**

Össur offers accessories that meet specific needs of the Unloader® One. These accessories can be purchased through the brace provider or Össur Customer Service Departments.

- Extra Set of Liners
- Replacement Straps
- Strap pads
- Undersleeves are NOT recommended
- Anti migration strap

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Unloader® is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.