Instructions for use

SmartSling®

Life Without Limitations®

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	7
ⓕⓗ Guide de fabrication.....	9
ⓔⓈ Instrucciones para el uso.....	11
ⓔⓣ Istruzioni per l'uso.....	13
Ⓝⓞ Bruksanvisning.....	15
ⓓⓐ Brugsanvisning.....	17
Ⓢⓕ Bruksanvisning.....	19
ⓕⓙ Käyttöohjeet.....	21
Ⓝⓕ Gebruiksaanwijzing.....	23
ⓔⓣ Instruções de Utilização.....	25

EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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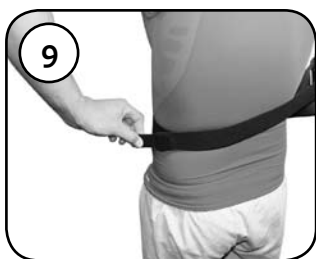
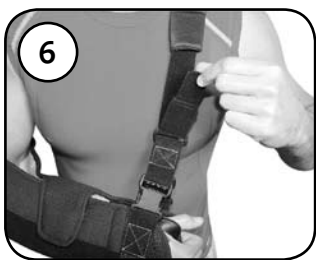
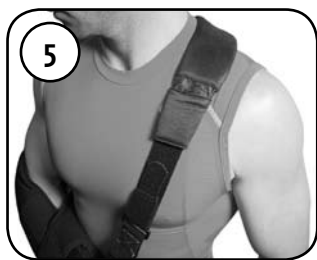
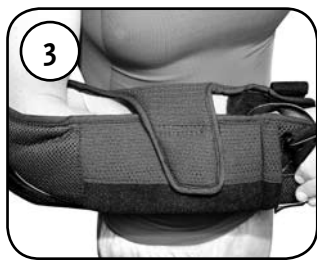
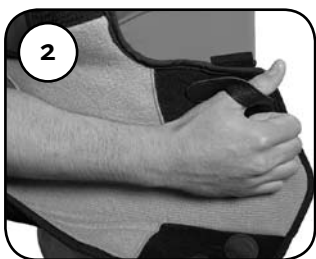
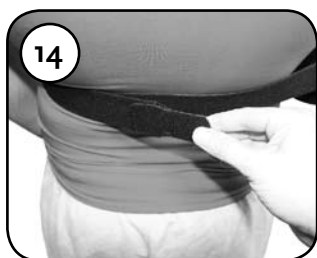
ENGLISH

WARNING

This sling is supportive only and is not intended or guaranteed to prevent injury. Purchaser agrees to the following:

- Ossur accepts no responsibility for any injury which might occur while using Ossur products.
- With use of this type of device, it is possible to receive various types of skin irritation, including rashes and pressure points. If you experience any unusual reactions while using this product, consult your health care provider immediately.
- If you have any questions as to how this product applies to your own specific condition, consult your health care provider.
- **If you do not agree to the above conditions, contact your health care provider before using this product.**

GENERAL APPLICATION

- 1 Place the forearm into the SmartSling at a 90° angle, seating the elbow snugly in the corner pocket of the sling. The installed elbow pillow can be removed, if necessary, to improve the fit and comfort.
- 2 Place the thumb through the thumb loop. The thumb loop can be repositioned for an improved fit and increased comfort.
- 3 Close the top flap to secure the sling.
Note: right arm application is shown in the photo. For left arm application, the flap will fold towards the body.
- 4 Place the adjustable strap around the neck and fasten it to the sling via the quick-release buckles.
- 5 Position the gusseted chamois neck pad where it is most comfortable.
- 6 If necessary, adjust the shoulder strap from the front (shown here) or back (not shown). If the strap is too long, consult the section on **Trimming Straps**.

IMMOBILIZATION

- 7 Locate the pocket on the side of the sling facing towards the body. Connect the immobilizer strap to the buckle located in the pocket.
- 8 Pull the immobilizer strap around the back and feed it through the loop located on the elbow end of the sling.
- 9 Pull the immobilizer strap back in the opposite direction —on top of itself — and fasten it snugly to itself. If the strap is too long, consult the section on **Trimming Straps**.

ABDUCTION

- 10 Attach one of the included double-sided hook strips to the loop material on the side of the sling facing the body.
- 11 Position the abduction pillow between the body and the sling, using the hook strip installed in step #1 to connect the sling to the pillow.
- 12 If desired, the underarm pillow can be used to provide additional support.
- 13 Pull the abduction pillow strap around the back and attach it to the abduction pillow via the quick-release buckle located on the front of the pillow.
- 14 The abduction pillow strap can be adjusted to obtain the desired fit. If the strap is too long, consult the section on **Trimming Straps**.

TRIMMING STRAPS

- 15 All straps on the SmartSling feature alligator clips for easy strap trimming. If any strap is too long, simply remove an alligator clip, trim the strap to the desired length and replace the clip.

WASHING & CARE

The SmartSling may be cleaned by hand using a mild soap and lukewarm water. Hang to dry.
Do not put the SmartSling in a washing machine or dryer.

LIMITED WARRANTY

This warranty covers all defects in materials and workmanship or malfunctions of your new SmartSling for 90 days from the date of original purchase.

Ossur will repair or replace, at its discretion, any defective or malfunctioning part covered under warranty at no charge. Repairs necessitated by accident, negligence, misuse or abuse by the purchaser are not covered. Also, consequential and incidental damages are not recoverable under this warranty. These limitations may not apply as some states do not allow the exclusion or limitation of incidental or consequential damages.

To obtain service under this warranty, please contact the Inside Sales Department of Ossur at (800) 222-4284 to obtain a Return Goods Authorization Number. Once you have obtained the Return Goods Authorization Number, please send your SmartSling (postage prepaid) with a brief written description of the problem to: Ossur, 19762 Pauling, Foothill Ranch, CA 92610. **Please write the Return Goods Authorization Number on the outside of the package.** Any product received without a Return Goods Authorization Number will be subject to a 2-week delay. If the problem is covered by the warranty, Ossur will promptly repair the product and return it to you. If we determine that the problem is not covered by the warranty, Ossur will so advise you and offer to remedy the problem, if possible, by providing you with an estimate of costs for parts and labor. Repairs authorized by the original purchaser will be made and the product returned with costs billed either COD or to a credit card. You must pay any charges due upon receipt of the repaired product. If repairs are not authorized by the original purchaser, Ossur will return the product to you in the same condition as it was received. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

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