Instructions for use

Form Fit[®] Thumb Spica

Life Without Limitations[®]

.....	3
ⓔⓃ Instructions for use.....	5
ⓓⓔ Gebrauchsanweisung.....	7
ⓕⓗ Guide de fabrication.....	9
ⓔⓢ Instrucciones para el uso.....	11
ⓔⓣ Istruzioni per l'uso.....	13
Ⓝⓞ Bruksanvisning.....	15
ⓓⓐ Brugsanvisning.....	17
Ⓢⓕ Bruksanvisning.....	19
ⓕⓙ Käyttöohjeet.....	21
Ⓝⓕ Gebruiksaanwijzing.....	23
ⓔⓣ Instruções de Utilização.....	25

EN- This product has been designed and tested based on single patient usage. It is a single-use device and should never be used by multiple patients. If any problems occur with the use of this product, immediately contact your medical professional.

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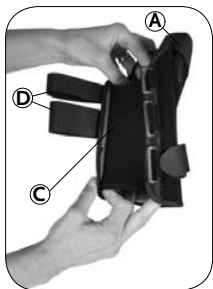
ENGLISH

USES

Gameskeeper's Thumb, selected scaphoid injuries, soft tissue injuries, DeQuevain's Tendinitis, and post cast healing.

CAUTION

Patients should be instructed to inspect their hand, wrist, thumb, and product before and after each use.

PHOTO CAPTIONS

- A. Thumb support
- B. Circumferential strap d-ring
- C. Stockinet
- D. Contact closure straps
- E. Thumb stay
- F. Palmar stay
- G. Circumferential strap

FITTING INSTRUCTIONS

1. Undo contact closure straps and circumferential strap.
2. Place hand into the spica and thumb into thumb support. Ensure stockinet positions on top of the hand.
3. Align the palmar stay to the palm of the hand. Adjustments on the aluminum palmer stay can be performed by a certified medical professional.
4. Ensure the thumb stay is on the radial border of the forearm. Adjustments on the aluminum thumb stay can be performed by a certified medical professional.
5. Ensure that the adjustable circumferential strap loops through both d-rings for added support. Fasten circumferential strap.
6. Fasten all contact closure straps and secure to a snug fit. Adjustments may be necessary as hand, wrist, and thumb swelling patterns change. Straps may be cut as desired.

HOT/COLD THERAPY

1. For cold therapy, remove gel pad from brace and place into a freezer for approximately 20 minutes to 1 hour. To reduce swelling, apply directly to the skin with hand elevated for 20 minutes.
2. For hot therapy, remove gel pad from brace and place in microwave oven for 20 second intervals at low setting until hot. Do not use oven or flame to heat the gel pad. Do not apply heated gel pad directly to skin. If too hot, allow to cool before applying.
3. Once gel pad is hot/cold, use contact closure tabs to adhere it into the brace to continue therapy. Diabetics and patients with vascular insufficiencies must always consult a physician before using hot/cold therapy.

WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Remove aluminum stays and gel pad before washing. Do not use bleach or strong detergents. Do not wash in washing machine.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater.

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Form Fit® Thumb Spica is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.